# Interconnect Tubing for NIBP Cuffs

*Instructions for Use*

*Tubing for Neonatal Cuffs*

**REF** M1596B 1.5m Tubing

**REF** M1597B 3.0m Tubing

*Tubing for Adult/Pediatric Cuffs*

**REF** M1598B 1.5m Tubing

**REF** M1599B 3.0m Tubing

# PHILIPS

---

# About This Edition

## Notice

### Copyright © 2007

Koninklijke Philips Electronics N.V.  
All rights are reserved.

## Printing History

### Current Edition/Revision:

Printed in Germany December 2007  
Part Number 453564077031

## Contents

Instructions for Use.....	1	Návod na používanie.....	29
Manuel d'utilisation.....	3	Navodila za uporabo.....	31
Gebrauchsanweisung.....	5	Instrucțiuni de utilizare.....	33
Gebruiksaanwijzing.....	7	Указания за употреба.....	35
Instrucciones de uso.....	9	Kasutusjuhend.....	37
Instruções de Uso.....	11	Naudojimo instrukcija.....	39
Istruzioni d'uso.....	13	Lietošanas instrukcijas.....	41
Brakerhåndbok.....	15	Инструкция по эксплуатации.....	43
Bruksanvisning.....	17	Οδηγίες Χρήσης.....	45
Käyttöohjeet.....	19	Kullanim Talimatları.....	47
Brugerhåndbog.....	21	ユーズ・ガイド.....	49
Használati útmutató.....	23	使用説明.....	51
Instrukcja obsługi.....	25	使用说明书.....	53
Návod k obsluze.....	27		

## USA Restriction

United States law restricts this device to sale by or on the order of a physician.

---

## Warnings

- Never connect intra-arterial or intra-venous lines, or any other Luer connectors, to the NIBP tubing. This can cause serious injury or death.
- These products are for use only in combination with Philips Patient Monitors when listed as accessories in the monitor's user documentation.
- The connector requires regular lubrication. Lubricate when cuffs become difficult to connect and disconnect (spray type silicone release agent and lubricant is recommended). Ensure lubricant does not enter the tube itself.

---

## Cleaning

To keep tubing free of dust and dirt, clean with a lint-free cloth, moistened with either soap and warm water (40° C/104° F maximum), a diluted non-caustic detergent, or one of the approved cleaning agents listed below. Do not allow sustained contact with cleaners—remove immediately by wiping tube with a clean, water-dampened cloth. Never immerse or soak in any liquid. Fluid in tubing will damage monitoring equipment.

Deteriorated or damaged tubing should be replaced. Do not continue use for patient monitoring.

<b>Recommended Cleaning Agents and Brands</b>	
<b>Soaps</b>	mild soaps
<b>Tensides</b>	dishwasher detergents: Edisonite Schnellreiniger <sup>®</sup> , Alconox <sup>®</sup>
<b>Ammonias</b>	Dilution of Ammonia <3%, Window cleaner
<b>Alcohol based</b>	Ethanol 70%, Isopropanol 70%, Window cleaner

---

## Disinfecting

Prior to disinfecting, tubing should first be cleaned (see *Cleaning*). To avoid long term damage to the tubing, disinfect only when necessary as specified by your facility. Do not allow sustained contact with disinfectants—remove immediately by wiping tube with a clean, water-dampened cloth. Never immerse or soak in any liquid. Fluid in tubing will damage monitoring equipment.

Recommended Disinfecting Substances	
Alcohol based	Ethanol 70%, Isopropanol 70%
Aldehyde based	Cidex <sup>®</sup>

---

## Sterilizing

**CAUTION: Do not autoclave the tubing or use bleaches containing sodium hypochlorite (e.g., Clorox<sup>™</sup>).**

Prior to sterilizing, tubing should first be cleaned (see *Cleaning*). To avoid long term damage to the tubing, sterilize only when necessary as specified by your facility.

The tubing has been tested to withstand Ethylene Oxide (EtO) gas sterilization. Be sure that all safety precautions regarding aeration after EtO exposure are followed.

---

## Ordering information

The tubing is for use with Philips monitoring equipment only:

- neonatal NIBP cuffs: M1596B (1.5 m) and M1597B (3.0 m)
- adult and pediatric cuffs: M1598B (1.5 m) and M1599B (3.0 m)

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For more information, please call your local Philips sales office listed in your telephone directory or a Philips regional office listed below for the location of your nearest sales office.

**United States:**

Philips Medical Systems  
North America Corporation  
3000 Minuteman Road  
Andover, MA 01810-1099  
(800) 225-0230

**Canada:**

Philips Medical Systems Canada  
281 Hillmount Road  
Markham, ON  
L6C 2S3  
(800) 291-6743

**Latin America Headquarters:**

Philips Medical Systems  
1550 Sawgrass Corporate Parkway #300  
Sunrise, FL 33323 USA  
Tel: (954) 835-2600  
Fax: (954) 835-2626

Philips Medical Systems  
3000 Minuteman Road  
Andover, MA 01810-1099  
USA  
(800) 225-0230

**Europe, Middle East and Africa:**

Philips Medizin Systeme  
Böblingen GmbH  
Hewlett-Packard Str. 2  
71034 Böblingen, Germany  
Fax: (+49) 7031 463 1552

**Asia Pacific Headquarters:**

Philips Medical Systems  
30/F Hopewell Centre  
17 Kennedy Road  
Wanchai  
Hong Kong  
Tel: (852) 2821 5888  
Fax: (852) 2527 6727

*<http://shop.medical.philips.com>*

**GMDN: 15640**

**Manufactured in Germany**

Subject to modification  
Copyright © 2007  
Koninklijke Philips Electronics N.V.  
Printed in Germany December 2007  
Part No. 453564077031

\*+H1794535640770313.\*